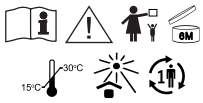
Intensive moisturizing eye drops

Čeština
Deutsch
Ελληνικά
English
Español
Français

Italiano
Magyar
Nederlands
Polski
Português
Slovenčina
Svenska

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English
What is it and what it is intended for This is an intensive moisturizing eye drop intended to relieve moderate and severe dry eye symptoms. The preservative free eye drops suit also sensitive eyes and contact lens users. The combined action of trehalose and hyaluronan provides hydration, lubrication and stabilization of the tear film,

protecting the eye from drying and cell damage. Regular use relieves dry eye symptoms – Itching, burning and foreign body sensation in the eyes – caused by environmental factors such as dry air, wind, smoke, air pollution, excessive light, visual display work or contact surgery and contact lens wear. The intensive moisturizing eye drops suit long-term and frequent use. This is a medical device developed and manufactured in Finland.

Instructions for use
Practise handling of the bottle before the first application. Apply 1–2 drops in the eyes 1–3 times daily, as needed. Remove the seal of the cap when opening the bottle for the first time. Remove the cap for application and put it immediately back on after use. Squeeze the bottle only in tip down position. You can use moisturizing drops during contact lens wear.

Precautions and warnings
• Do not use if you are sensitive to any of the ingredients. • Do not use if the bottle is damaged. • Each bottle should be used by one user only. • Do not touch the eye or contact lens with the tip of the bottle. Do not touch the tip of the bottle with fingers. • Do not squeeze the bottle with the cap on. • Do not use after the expiry date or after discard date (6 months from opening). • Mild temporary blurring of vision may be experienced immediately after application. It is therefore advisable to wait a few seconds before continuing any activity that requires clear vision. • Leave at least 10–15 minutes between applications of any other ophthalmic preparation. • Keep out of the reach and sight of children. • Store at 15–30 °C, protect from direct sunshine and freezing. • Discard the empty bottle appropriately.

Composition
2.0% trehalose, 0.25% hyaluronan in sterile isotonic, low-phosphate-buffered solution. No materials of animal origin.

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**Jaakkoo-Taara Oy**